


How to use the single use distal cover  
MAJ-2315 for EVIS EXERA III Duodenovideoscope TJF-Q190V


**WARNING** This quick reference guide is a summary of the steps necessary to properly use your MAJ-2315. Follow the detailed steps instructed in the latest OPERATION/REPROCESSING MANUAL for TJF-Q190V.

**CAUTION** Do not apply anti-fogging products, olive oil, or products containing petroleum-based substances (e.g., Vaseline®) to the distal cover and the endoscope. These products may cause cracks of the distal cover. If a distal cover with cracks is used, it may cause thermal injury due to electric current leaks when high-frequency cauterization treatment is performed.



Attaching the distal cover

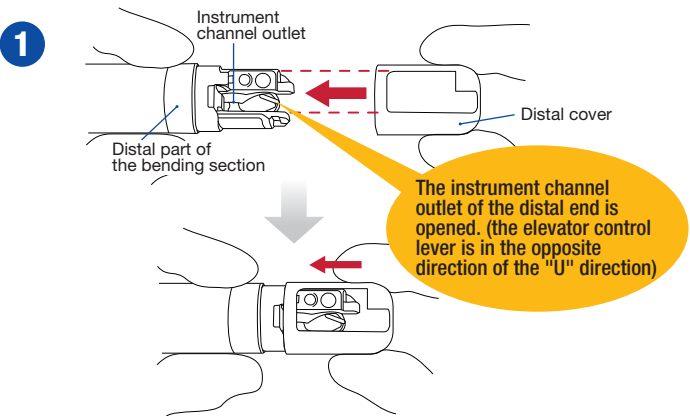

1. Gently hold the distal part of the bending section and the distal cover. Align the opening side of the distal cover with the lens side of the distal end of the endoscope.

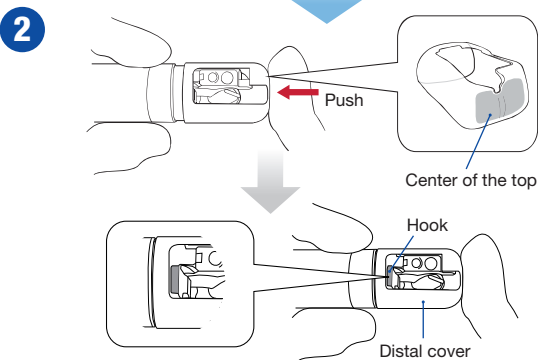

2. Put your finger onto the center on the top of the distal cover and push the top of the distal cover straight onto the distal end of the endoscope until the hook of the distal ring is completely visible within the opening of the distal cover.

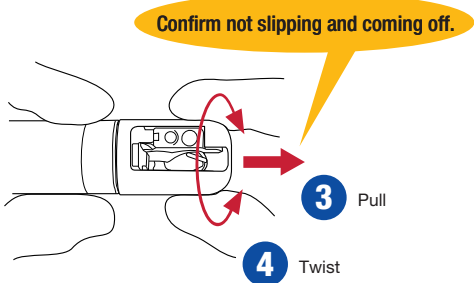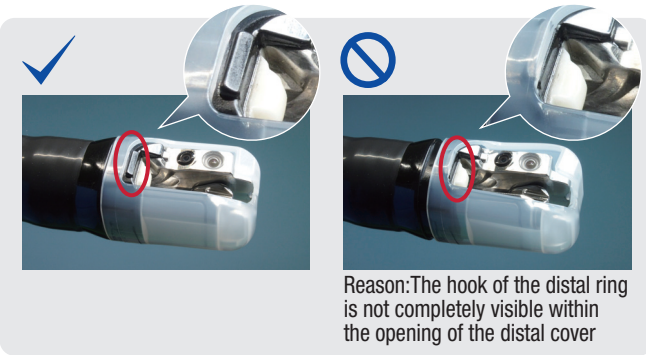

3. Hold the distal part of the bending section. Pull the distal cover gently to confirm that it does not slip and come off.


4. Twist the distal cover gently in both directions and confirm that the distal cover on the distal end of the endoscope does not slip and come off.




5. Confirm that the distal cover is free of cracks or deformation.

6. Confirm that there is no adhesion of foreign materials between the distal end of the endoscope and the distal cover.


7. Confirm that the single use distal cover is not seen on the endoscopic image.

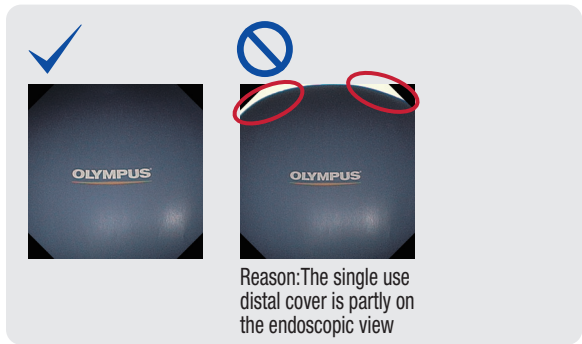


Detaching the distal cover

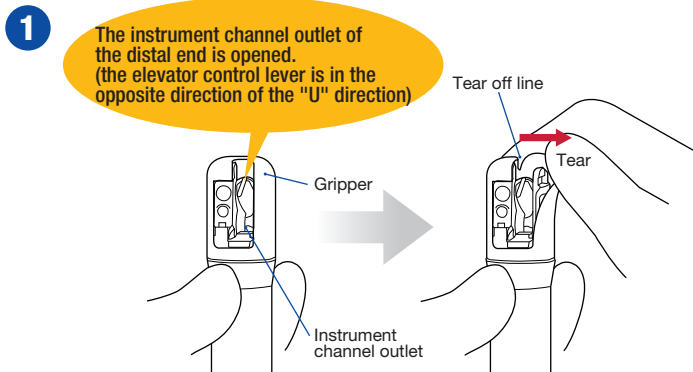

1. Hold the gripper of the distal cover and tear the top of the distal cover along the tear off line.

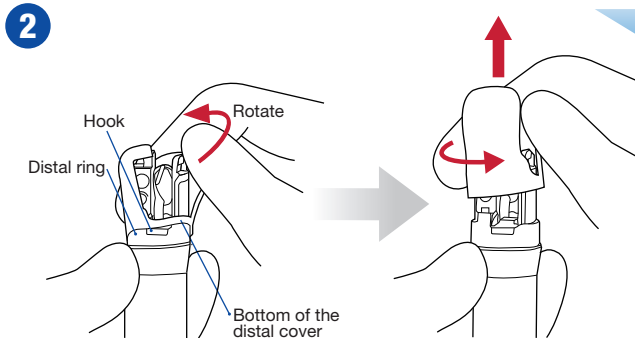

2. Keeping the gripper apart from the distal end, rotate the distal cover in the direction of the arrow as shown above until the bottom of the distal cover is removed from the hook of the distal ring to detach the distal cover.

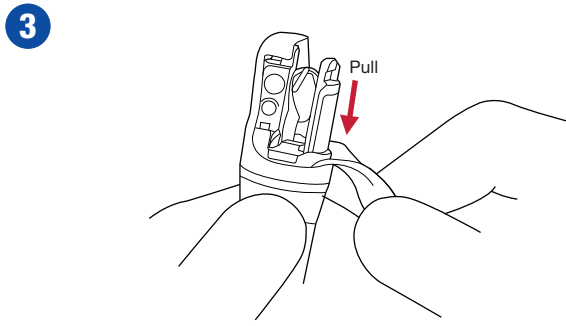

3. If the distal cover cannot be detached, pull the gripper toward the direction of the arrow as shown above and completely tear the distal cover from the distal end.



4. After using the distal cover, dispose of it in an appropriate manner.

Manufactured by  
**OLYMPUS MEDICAL SYSTEMS CORP.**  
2951 Ishikawa-cho, Hachioji-shi, Tokyo 192-8507, Japan Fax: (042)646-2429 Telephone: (042)642-2111

Distributed by

<b>OLYMPUS EUROPA SE &amp; CO. KG</b> (Premises/Goods delivery) Wendenstrasse 14-18, 20097 Hamburg, Germany (Letters) Postfach 10 49 08, 20034 Hamburg, Germany Fax: (040)23773-4656 Telephone: (040)23773-0	<b>OLYMPUS AMERICA INC.</b> 3500 Corporate Parkway, P.O. Box 610, Center Valley, PA 18034-0610, U.S.A. Fax: (484)896-7128 Telephone: (484)896-5000	<b>OLYMPUS AUSTRALIA PTY LTD</b> 3 Acacia Place, Notting Hill, VIC 3168, Australia Fax: (03)9543-1350 Telephone: (03)9265-5400
<b>KEYMED (MEDICAL &amp; INDUSTRIAL EQUIPMENT) LTD.</b> KeyMed House, Stock Road, Southend-on-Sea, Essex SS2 5QH, United Kingdom Fax: (01702)465677 Telephone: (01702)616333	<b>OLYMPUS LATIN AMERICA, INC.</b> 5301 Blue Lagoon Drive, Suite 290 Miami, FL 33126-2097, U.S.A. Fax: (305)261-4421 Telephone: (305)266-2332	<b>OLYMPUS SINGAPORE PTE LTD</b> 438B Alexandra Road #03-07/12, Alexandra Technopark Bldg B, Singapore 119968 Fax: 6834-2438 Telephone: 6834-0010
<b>OLYMPUS MOSCOW LIMITED LIABILITY COMPANY</b> Elektrozavodskaya str. 27 bld 8, 107023 Moscow, Russia Fax: (7495)926-7072 Telephone: (7495)926-7077	<b>OLYMPUS (BEIJING) SALES &amp; SERVICE CO., LTD.</b> A8F, Ping An International Financial Center, No. 1-3, Xinyuan South Road, Chaoyang District, Beijing, 100027 P.R.C. Fax: (86)10-5976-1299 Telephone: (86)10-5819-9000	<b>OLYMPUS KOREA CO., LTD.</b> 2F-3F Majestiar City Tower One, 12, Seocho-daero 38-gil, Seocho-gu, Seoul, 06655, Korea Fax: (02)6255-3210 Telephone: 1544-3202