


LithoCrush V

Single Use Mechanical Lithotripter V

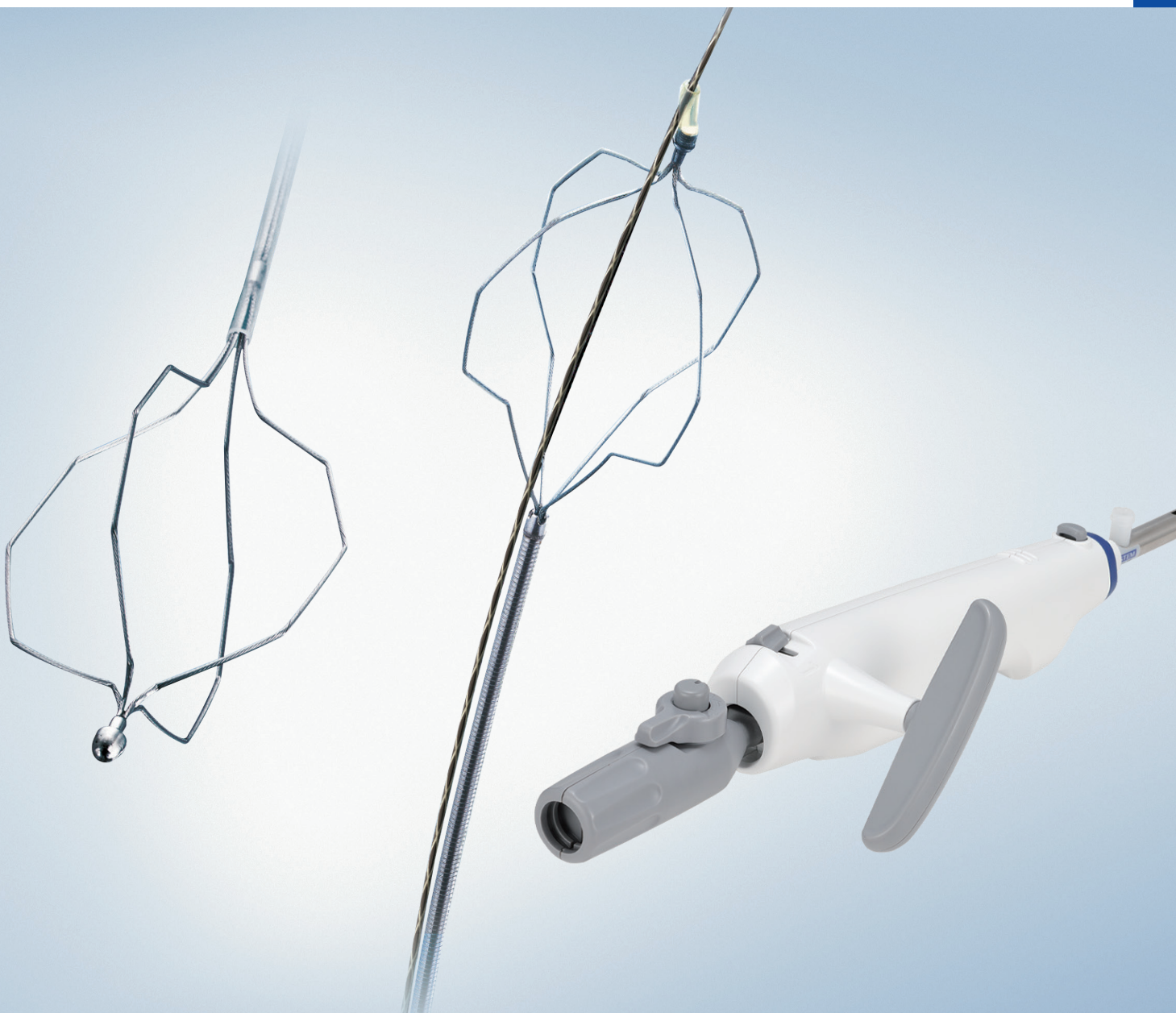


New Lighter Single-use Handle

Has Same Crushing Ability as Olympus Reusable Metal Handle

Easy Setup by Lock/unlock Symbol and Clicking

LithoCrushV baskets are secured into LithoCrushV handle with an audible, one-touch locking mechanism.

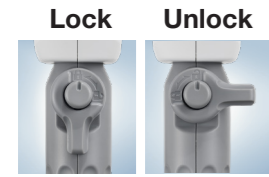

Ratchet Prevents Loss of Traction on the Stone

The ratchet prevents counter-rotation that can occur when a calculus is crushed.

Ready-to-Use Sterile Package

Single-use design eliminates the inconvenience of reprocessing and helps mitigate the risk of infection.

Same Crushing Ability and Rotation Function as Olympus Reusable Metal Handle


The contoured handle and T-Lever are easy to grip allowing user to apply pressure and control the basket during lithotripsy. All LithoCrushV baskets feature a rotation mechanism that enables efficient capture of small stones and crushed stone fragments.

The Easy-to-use Rotatable LithoCrush V

Features Distal Wireguided and Bullet Tip Design Options to Fit the Needs of Virtually Every Procedure


Bullet Tip Design Provides Powerful Crushing Performance

With a dual tube and coil sheath construction, the bullet tip design combines outstanding insertion ability with remarkable torque performance.

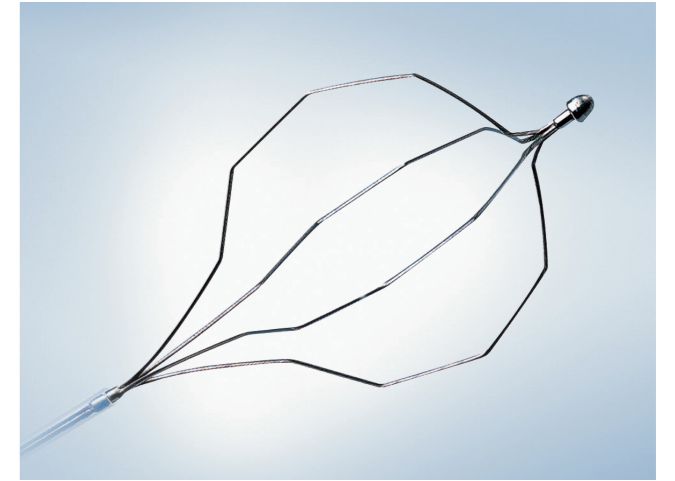

Distal Wireguided Design Improves Insertion Capability

Distal wireguided design makes it easier to exchange catheters and insert the basket into the papilla. It also facilitates selective cannulation of the intrahepatic bile ducts or other target sites.

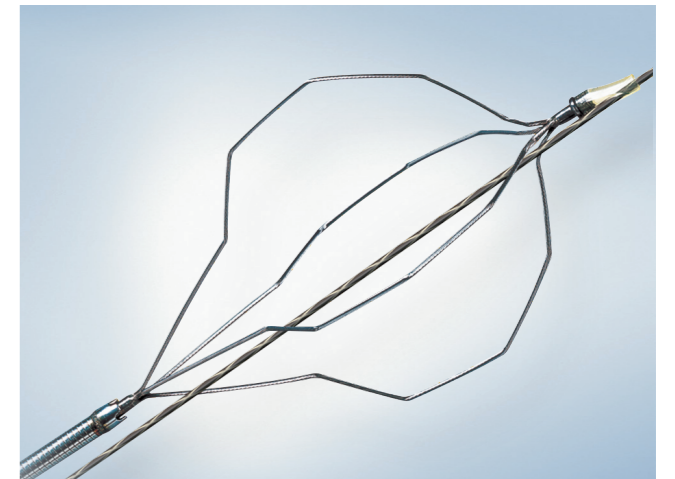


Distal wireguided construction
With the distal wireguided style models, the rotation mechanism functions only when the guidewire is pulled out.

Extra Strong Basket Available for Enhanced Crushing Power


The hard-wire type basket with 19 strands versus the standard 12 strands has greater crushing power and is ideal for repeated crushing.

Wire-constructed cross section

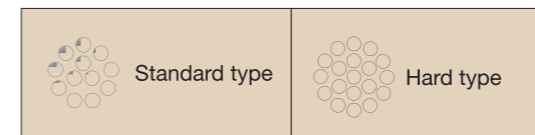

Convenient Injection Port

An injection port allows for injection of contrast medium with a syringe.

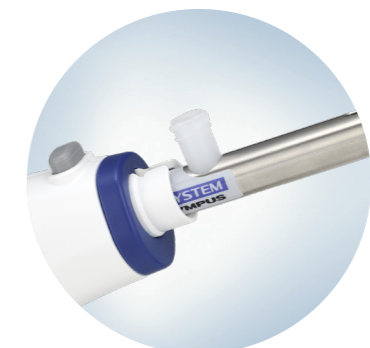

LithoCrush V

Single Use Mechanical Lithotripter V

Specifications


Single Use Mechanical Lithotripter V

Model	Bullet Tip/Wireguided Type	Opening Width	Hard/Standard Basket-wire Type	Minimum Channel Size	Compatible Guidewire
BML-V242QR-30	Bullet tip type	30 mm	Hard	4.2 mm	-
BML-V232QR-30	Bullet tip type	30 mm	Standard	3.2 mm	-
BML-V232QR-26	Bullet tip type	26 mm	Standard	3.2 mm	-
BML-V442QR-30	Distal wireguided type	30 mm	Hard	4.2 mm	0.89 mm (0.035")

: Refer to instructions.

LithoCrush V handle

Model	Feature
MAJ-2502	Single-use, rotation mechanism, ratchet prevents counter-rotation

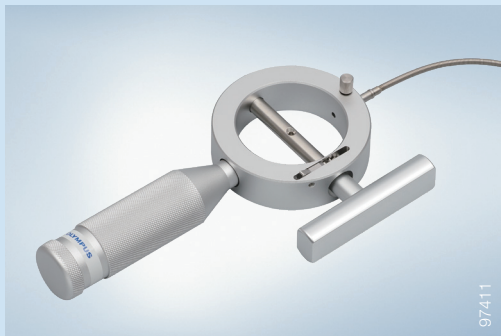

Single Use Mechanical Lithotripter BML-610A

In the unlikely event of basket impaction, the BML-610A Emergency Lithotripter Handle is to be used.

Olympus EndoTherapy

Dedicated Support beyond Products

At Olympus, we strive to be more than just a medical-equipment provider to our customers. We provide end-to-end support, from the purchasing process to procedure and reprocessing services, to build a relationship of trust.



ERCP Showroom Page

Enter the world of ERCP on our Olympus website for a full overview of our product portfolio, valuable video content, information on training and much more.

www.olympus.eu/ercp-procedure



Training

Register here to improve procedure skills and the safe and effective use of Olympus products.

www.olympus.eu/training



Portfolio

Browse through the rich portfolio of EndoTherapy devices here.

www.olympus.eu/et-catalog



Service

Contact your Olympus sales representative for individual support now.

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.