


EVIS EUS

EVIS EXERA II ULTRASOUND GASTROVIDEOSCOPE

GF-UCT180

Offering Increased Imaging Depth, This Scope's New Curved Linear Array Transducer Lets You Step Up to the Next Level in Ultrasound Imaging and Performance

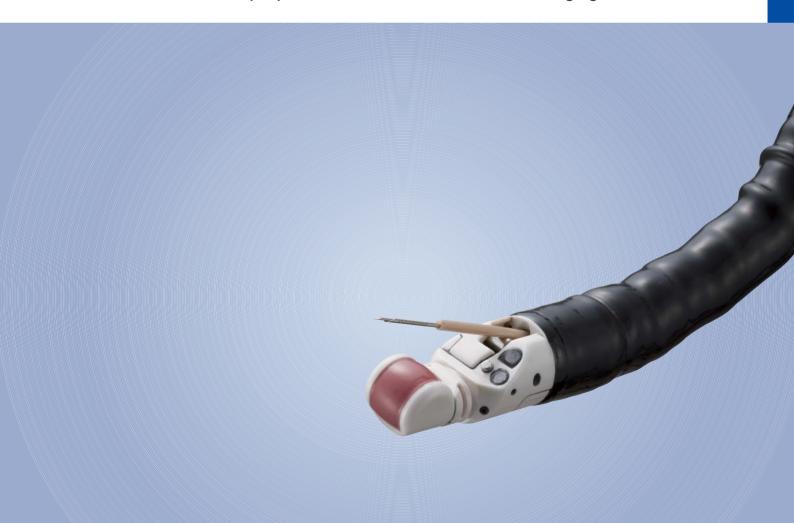


Improved B-mode imaging capability, redesigned forceps elevator, Contrast Harmonic Echo capability*, and detachable ultrasound cable




Featuring a newly developed curved linear array transducer that delivers higher-quality ultrasound images with greater imaging depth in B-mode, the GF-UCT180 offers you the power and flexibility you need for everything from straightforward tissue sampling to sophisticated EUS-guided treatment. A redesigned forceps elevator gives you more reliable control over devices passed through the 3.7 mm diameter instrument channel, while the slim 12.6 mm diameter insertion tube ensures smooth insertion. Advanced capabilities include support for the Contrast Harmonic Echo*, the technique that can realize visualization of microvasculature structure. Other features include a high-resolution CCD for sharp video images, Narrow Band Imaging (NBI) capability and a detachable ultrasound cable that facilitates easier handling, reprocessing and storage, as well as ensuring system compatibility.

*Available only in combination with the ALOKA ProSound α 10.

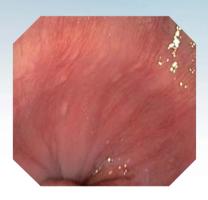



B-mode

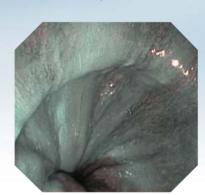

NBI



Color flow mode

The GF-UCT180 offers outstanding platform flexibility using a single detachable cable that lets you connect the ultrasound endoscope to a choice of OLYMPUS and ALOKA ultrasound centers



UNIVERSAL ENDOSCOPIC ULTRASOUND CENTER

EU-ME1

Integrates electronic and mechanical scanning in a single unit

Fitting comfortably on the shelf of an endoscopic trolley, this extraordinarily versatile unit is fully compatible with the complete OLYMPUS line of ultrasound endoscopes, regardless of whether the connected scope uses mechanical or electronic EUS technology, curved linear array or radial scanning mechanisms. With an excellent B-mode image, highly sensitive Power and Color Doppler capability, and more, the EU-ME1 offers exceptional performance in an integrated compact design.

Hitachi ALOKA Medical Diagnostic Ultrasound System

ProSound α 7

Unparalleled performance of high-end model and enhanced user, patient and environmental friendliness are packed in the compact ProSound $\alpha 7.$ The Broadband Harmonics realizes high sensitivity and resolution for Tissue Harmonic Echo. The superb eFLOW features enhanced spatial resolution, visualizing blood flow dynamics ranging from thin and slow flows to thick and fast flow in greater detail. Optional wireless remote controller is available. It has a wide variety of compatibility with OLYMPUS ultrasound scopes and allows for excellent ultrasound performance with GF-UCT180 and subsequent ultrasound scopes.

GF-UCT180 supports a variety of procedures from imaging EUS to interventional EUS



Detachable ultrasound cable

A detachable ultrasound cable design makes the GF-UCT180 easier to handle, reprocess and store and provides it with the flexibility to connect to different ultrasound platforms.


Redesigned forceps elevator

The GF-UCT180 features a redesigned forceps elevator that gives you more reliable control over devices passed through the 3.7 mm diameter instrument channel.

Ideal for EUS-guided FNA

Combining the GF-UCT180's slim 12.6 mm diameter insertion tube, ensuring smooth insertion, with the EZ Shot 2 aspiration needle (NA-220H/230H) and connecting it to either platform shown above creates a system ideal for EUS-guided FNA.

OLYMPUS GF TYPE UCT180

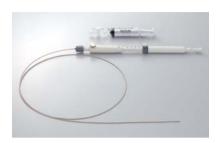

Single Use Aspiration Needle NA-220H/230H

EZShot 2...

Single-use, easy-to-use needle with excellent puncture capability

For use with any OLYMPUS ultrasound endoscope designed for EUS-guided fine needle aspiration (FNA), the pre-sterilized, pre-assembled and ready-to-use EZ Shot 2 offers good needle tip visibility and adjustable needle projection, from 0 to 80 mm. Simple to attach and easy to use, the EZ Shot 2 provides everything you need for effective FNA in one package



Ultrasound Cable

MAJ-1597


The ultrasound cable for the GF-UCT180 is optionally available and is not provided with the scope. The cable's detachable design makes the ultrasound endoscope easy to handle, reprocess and store, as well as ensuring excellent platform compatibility.

Specifications

Endoscopic Functions	Optical System	Field of view	100°
		Direction of view	Forward oblique viewing 55°
		Depth of field	3 to 100 mm
	Insertion Tube	Distal end outer diameter	14.6 mm
		Insertion tube outer diameter	12 . 6 mm
		Working length	1250 mm
	Instrument Channel	Channel inner diameter	3.7 mm
		Minimum visible distance	6 mm
		Direction from which endotherapy accessories enter and exit the endoscopic image	
	Bending Section	Angulation range	Up 130°, Down 90° Right 90°, Left 90°
	Total Length	1555 mm	



Ultrasound Functions		With EU-ME1	With ALOKA α 7
	Cable	MAJ-1597	
	Cable Length	1500 mm	
	Operation Mode	B-mode, Color Flow mode, Power Flow mode	B-mode, M-mode, D-mode, Power Flow mode, Flow mode
	Scanning Method	Electronic curved linear array	
	Scanning Direction	Parallel to the insertion direction	
	Frequency	5, 6, 7.5, 10, 12 MHz	4, 6.67, 10, 13.3 MHz
	Scanning Range	180°	
	Contacting Method	Balloon method, Direct Contact method	

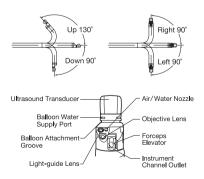

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.



OLYMPUS MEDICAL SYSTEMS CORP.

Shinjuku Monolith, 2-3-1 Nishi-Shinjuku, Shinjuku-ku, Tokyo 163-0914, Japan

For a complete listing of sales and distribution locations visit: www.olympus.com