



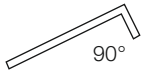


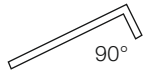

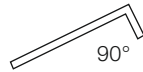


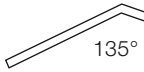
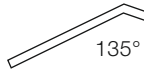
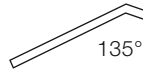
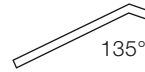


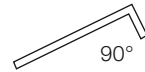

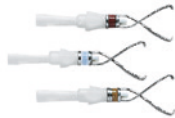





Clip Specifications				
Super Short (4 mm) / Short (6 mm) / Standard (7.5 mm) / Long (9 mm)				
Article Name	HX-610-090SC	HX-610-090S	HX-610-090	HX-610-090L
Article Number	N6015630	N6012530	N6012330	N6012730
Exterior of Jaw Angle				
Clip Arm Length	Short 	Short 	Standard 	Long 
Color Mode	 White	 White	 Yellow	 Blue
Quantity per Box	24	40	40	40

Article Name	HX-610-135XS	HX-610-135S	HX-610-135	HX-610-135L
Article Number	N6012930	N6012630	N6012430	N6012830
Exterior of Jaw Angle				
Clip Arm Length	Super Short 	Short 	Standard 	Long 
Color Mode	 Gray	 Green	 Pink	 Purple
Quantity per Box	24	40	40	40

Clip-Fixing Device Specifications						
Single-Use Applicator				Reusable Applicator		
						
Article Name	HX-810LR	HX-810QR	HX-810UR	HX-110LR	HX-110QR	HX-110UR
Article Number	N5432730	N5432830	N5432930	N1085260	N1085360	N1085460
Compatible Channel	2.8 mm or more			2.8 mm or more		
Working Length	1650 mm	1950 mm	2300 mm	1650 mm	1950 mm	2300 mm
Maximum Diameter	2.75 mm			2.7 mm		
Quantity per Box	10	10	10	1	1	1

> www.olympus.eu/et-catalog

EZ Clip

Multiple-Clipping System for Reliable Hemostasis



E0428333EN - 07/19 - OEKG


Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

Clips and Clip-Fixing Devices for Multiple Clipping

Olympus EZ Clip is a well-known system among healthcare professionals for quite some time now. And there’s a good reason for that. The system is highly flexible thanks to functions such as clip rotation and a versatile selection of clips to suit the requirements of any clinical situation. Additionally, it is a very cost-efficient system thanks to the unique reloadable design.


Clip Features

- Fast and effective mechanical hemostasis
- Eight types of EZ Clips to suit all applications
- Rotatable
- Sterile cartridge-housed clips
- 90° and 135° jaw angles available
- Short tail
- Colored cartridges to easily distinguish each clip
- For these clips, you have the choice between two different clip applicators

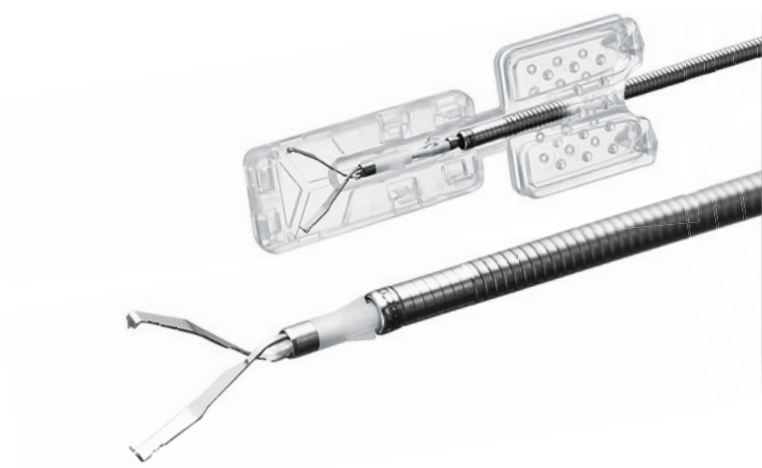

Easy Loading with Simple Push-Pull Action

The clips are housed in cartridges for fast, easy loading. All you have to do is insert the sheath into the cartridge, push the slider forward, and pull back.

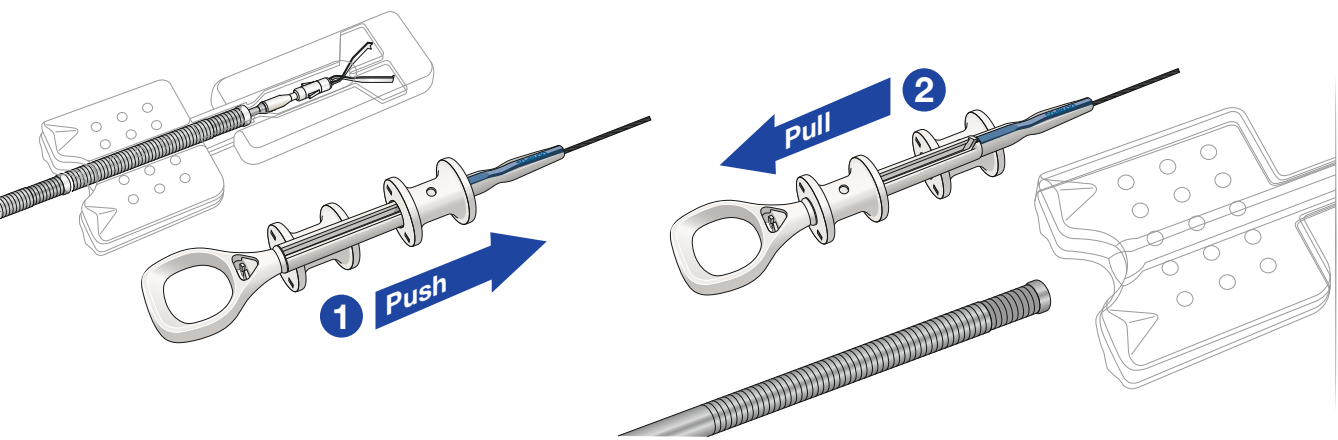

Clip Specifications				
Article Number	Clip Arm Length	Benefits		Jaw Angle
N6012930	Super short, approx. 4 mm	For hard tissue and bleeding ulcers		135°
N6012630	Short, approx. 6 mm	For stronger clipping/smaller area, hemostasis of stiffer lesions		135°
N6012430	Standard, approx. 7.5 mm	For pathological mucosa, deep ulcerous lesions, cancerous tissue		135°
N6012830	Long, approx. 9 mm	Wide clip opening, for post-EMR and post-polypectomy clipping in polyps with big stalk, for tangential approach		135°
N6012530	Short, approx. 6 mm	For marking		90°
N6012330	Standard, approx. 7.5 mm	For healthy, elastic mucosa; Mallory-Weiss syndrome; open vessels; small perforations; marking		90°
N6012730	Long, approx. 9 mm	For healthy, elastic mucosa; large-scale lesions; large adenomatous pedicles before and after polypectomy		90°
N6015630	Short, approx. 6 mm	Special clips for marking; colored distal part; 8 white, red, and yellow clips in each package		90°

Applicator Features

Easy clip reloading (push-pull action) with the clip-fixing devices makes it possible to perform multiple clippings smoothly and efficiently during one clinical case.

- Rotation function for easy targeting
- Compatible with all EZ Clip products (HX-610 series)
- Three working lengths available

Single-Use

- Eliminates the reprocessing procedure after use
- Cost-efficient single-use clip-fixing device for multiple clipping

Reusable

- Autoclavable

