


EndoPreZyme

Detergent for Manual Precleaning Prior to Automated Reprocessing



For Flexible Endoscopes, Endoscopic Accessories and Thermostable (Surgical) Instruments.
Suitable for Immersion and Ultrasonic Baths.

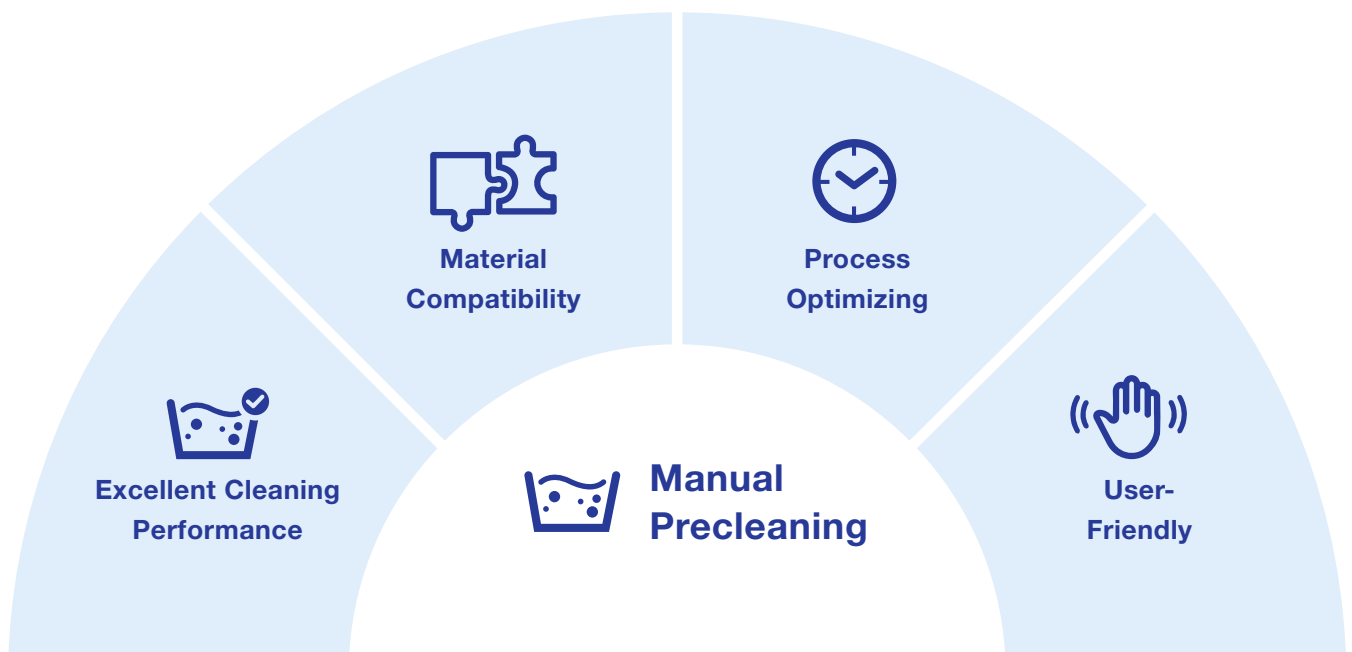

EndoPreZyme

Excellent Cleaning Performance

Proper cleaning is key for the reprocessing success. Due to the enzymatic formulation, Olympus EndoPreZyme provides an excellent cleaning performance prior to automatic reprocessing, which is supported by comprehensive testing. It removes soiling specific to endoscopy, including residues from X-ray contrast media, defoaming agents, throat anesthetics, lubricant gels and hemostasis sprays.

Material Compatibility

The material compatibility of the pH neutral detergent supports a longer lifetime of flexible Olympus endoscopes as well as all of other leading manufacturers.

Process Optimizing

EndoPreZyme optimizes the overall procedure of reprocessing by reducing the process time. The solution allows you to omit the rinse-off step prior to the automated reprocessing in an Olympus ETD machine.

User-Friendly

EndoPreZyme has a low-foaming formula, which provides you with good visibility of the instruments during the manual precleaning. The dosing amount can be adjusted (5-15 ml/l) according to the individual level of soiling of your instruments.



EndoPreZyme

Product Information

Dosage	5-15 ml/l (0.5-1.5%)*
Contact time	5-10 min.
Temperature	up to 35°C
pH Value	approx 7.1**
Shelf Life	2 years

* Water with a hardness of <3°dH is recommended; the hardness should not exceed 20°dH.

**Application solution in deionized water at a concentration of 5 to 15 ml/l)

Ordering Information

Item No.	WD00235A	WD00236A
Item Name	EndoPreZyme, 6 x 2 l	EndoPreZyme, 2 x 5 l
Contents	Detergent; pH neutral with enzymes; 6 x 2 l bottles	Detergent; pH neutral with enzymes; 2 x 5 l canisters

As medical knowledge is constantly growing, technical modifications or changes of the product design, product specifications, accessories and service offerings may be required.

DISTRIBUTOR

OLYMPUS

OLYMPUS EUROPA SE & CO. KG
Postbox 10 49 08, 20034 Hamburg, Germany
Wendenstrasse 20, 20097 Hamburg, Germany
Phone: +49 40 23773-0, Fax: +49 40 233765
www.olympus-europa.com

MANUFACTURER

Chemische Fabrik Dr. Weigert GmbH & Co. KG,
Mühlenhagen 85,
DE-20539 Hamburg, Germany