

ENDOSCOPE CLEANING GUIDE

for EVIS EXERA III Duodenovideoscope TJF-Q190V


WARNING



This workflow is a summary of the cleaning steps necessary to properly reprocess your TJF-Q190V and does not include disinfecting/sterilizing steps. Follow the detailed steps instructed in the latest ENDOSCOPE REPROCESSING MANUAL for TJF-Q190V.

* The handle of the single use combination cleaning brush (BW-412T) or the brush handle of the channel-opening cleaning brush (MH-507)
 ** The channel cleaning brush part of the single use combination cleaning brush (BW-412T) or the channel cleaning brush (BW-20T)

All steps of cleaning for endoscope are done in detergent solution or water completely, except for flushing with air and drying the external surfaces at the end of cleaning.

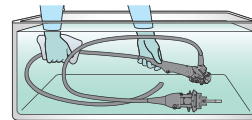


Clean the external surfaces



Brush the distal end of the endoscope

1 Detergent solution

Before "Cleaning the external surfaces", "Precleaning" and "Leakage testing" should be done as described in the reprocessing manual.

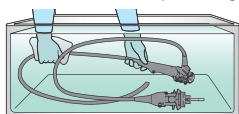


2

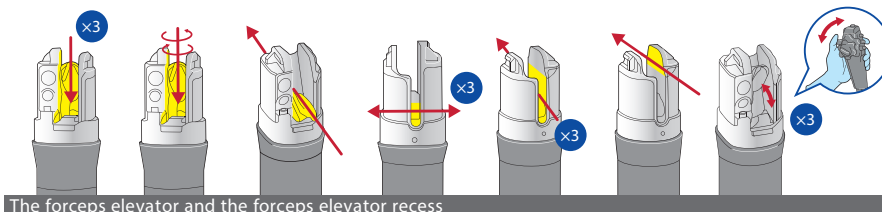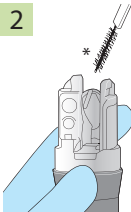

The forceps elevator and the forceps elevator recess


Brush the distal end of the endoscope

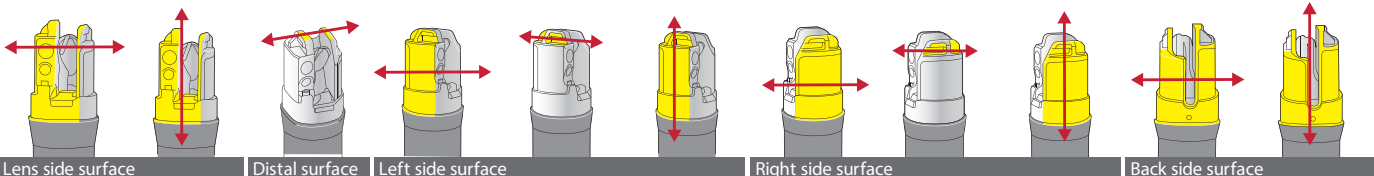

Brush the instrument channel and suction channel

Brush the suction cylinder & the instrument channel port

Flush the distal end

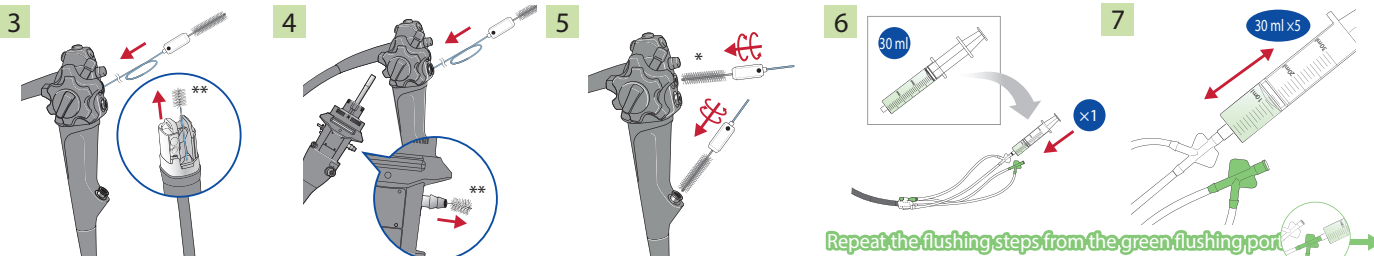


Flush the distal end

Aspirate detergent solution

Flush the air/water channel


Immerse in detergent solution

Remove detergent solution

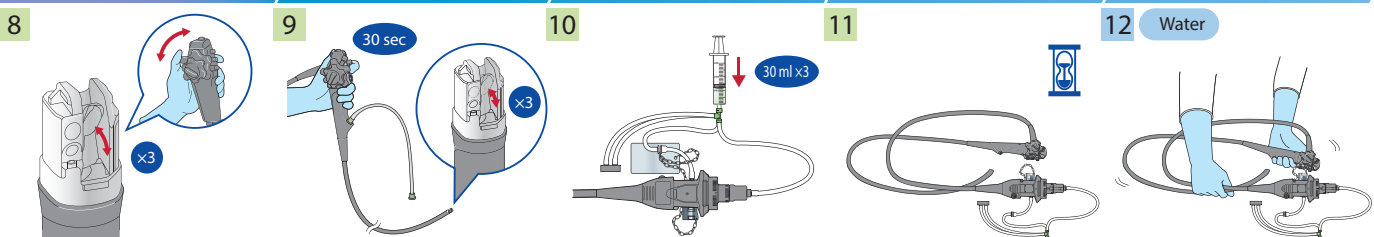

Remove detergent solution

Dry the external surfaces

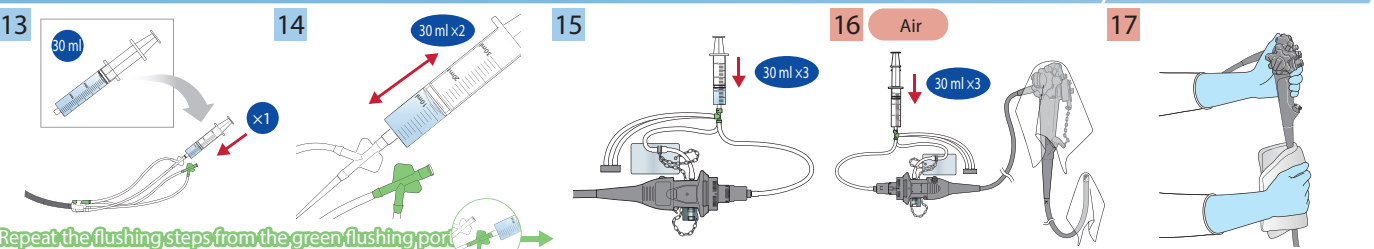

Manufactured by

OLYMPUS MEDICAL SYSTEMS CORP.
 2951 Hattokawa-cho, Hachioji-shi, Tokyo 192-8507, Japan
 Fax: (042)646-2429 Telephone: (042)646-2111

Distributed by

OLYMPUS EUROPA SE & CO. KG
 (Premises/Goods delivery) Wendenstrasse 14-18, 20097 Hamburg, Germany
 (Letters) Postfach 10 49 58, 20044 Hamburg, Germany
 Fax: (040)23773-4656 Telephone: (040)23773-0

KEYMED (MEDICAL & INDUSTRIAL EQUIPMENT) LTD.
 KeyMed House, Stock Road, Southend-on-Sea, Essex S52 5QH, United Kingdom
 Fax: (01702)465677 Telephone: (01702)616333

OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
 Elektrozavodskaya str. 27 bld.8, 107023 Moscow, Russia
 Fax: (7495)926-7072 Telephone: (7495)926-7077

OLYMPUS AMERICA INC.
 3500 Corporate Parkway, P.O. Box 610, Center Valley, PA 18034-0610, U.S.A.
 Fax: (484)966-7128 Telephone: (484)966-5000

OLYMPUS LATIN AMERICA, INC.
 5301 Blue Lagoon Drive, Suite 290 Miami, FL 33126-2097, U.S.A.
 Fax: (305)261-4421 Telephone: (305)266-2332

OLYMPUS (BEIJING) SALES & SERVICE CO., LTD.
 AIF, Ping An International Finance Center, No. 1-3, Xinyuan South Road,
 Chaoyang District, Beijing, 100027 P.R.C.
 Fax: (86)10-5976-1299 Telephone: (86)10-5819-9000

OLYMPUS AUSTRALIA PTY LTD
 3 Acacia Place, Notting Hill, VIC 3168, Australia
 Fax: (03)9543-1350 Telephone: (03)9543-5400

OLYMPUS SINGAPORE PTE LTD
 4388 Alexandra Road #03-0712, Alexandra Technopark Bldg B, Singapore 119968
 Fax: 6534-2438 Telephone: 6534-0010

OLYMPUS KOREA CO., LTD.
 2F-3F, Majestic City Tower One, 12, Seocho-daero 38-gil, Seocho-gu, Seoul,
 06655, Korea
 Fax: (02)9255-3210 Telephone: 1544-3202