ENDOSCOPE PRECLEANING GUIDE

EVIS (140/240), EXERA (160), EXERA II (180) & EXERA III (190) GI Endoscopes


— Refer to companion *ENDOSCOPE CLEANING GUIDE* for additional cleaning information —

WARNING: This guide is only a summary of the steps necessary to properly preclean your endoscope. Be sure to follow the detailed steps outlined in the *ENDOSCOPE REPROCESSING MANUAL* that was included with your endoscope when purchased and in the *ENDOSCOPE CLEANING GUIDE* for a summary of manual cleaning steps.

Be sure to follow precleaning with appropriate leakage testing, manual cleaning, and disinfection steps.


PRECLEANING – Immediately After Use

1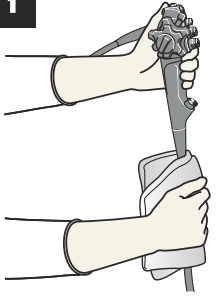


Wipe down insertion tube with a water-soaked cloth.

2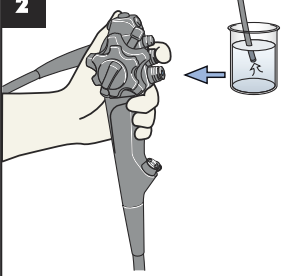

Aspirate water, then air through the suction channel.


3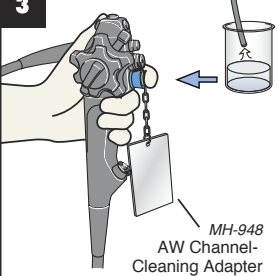

MH-948
AW Channel-Cleaning Adapter

Attach AW channel-cleaning adapter. Flushwater, then air, through air/water channels.


4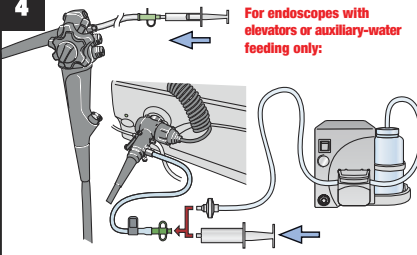

For endoscopes with elevators or auxiliary-water feeding only:


Flush water, then air, into elevator-wire channel/auxiliary-water channel. (Alternatively, the auxiliary-water channel may be flushed using the OFP pump.) The auxiliary water tube should remain attached during transport to reprocessing room.

5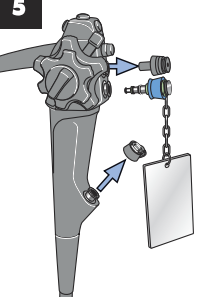

Disconnect all detachable parts.

6

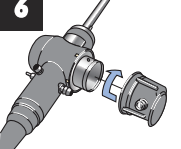Attach water-resistant cap. This step is omitted for 190 endoscopes. Transport to the reprocessing room in a covered container.

NOTE: Ensure that the ETO cap is removed prior to leak testing 190 generation scopes.

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