


VIDEO BRONCHOSCOPE CLEANING GUIDE

EVIS EXERA III and Earlier


WARNING: This workflow description is a summary of steps to properly pre-clean your EVIS EXERA III endoscopes. Follow detailed steps instructed in the latest ENDOSCOPE REPROCESSING MANUAL.

Precleaning at Bedside


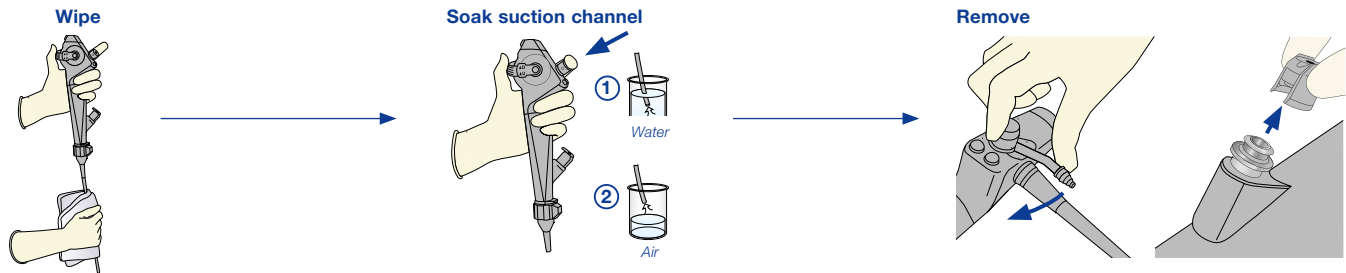


Leakage Testing


Connecting the EVIS EXERA II with the leakage tester

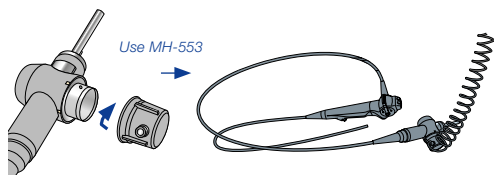

Connecting the EVIS EXERA III with the leakage tester

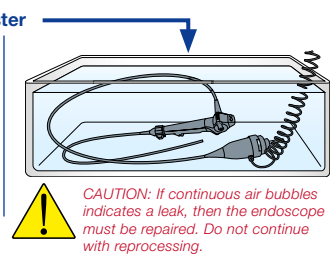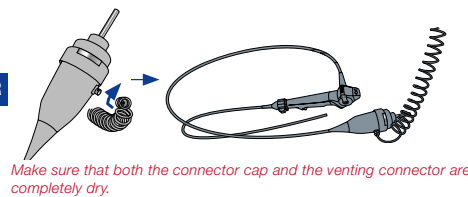


Manual Cleaning

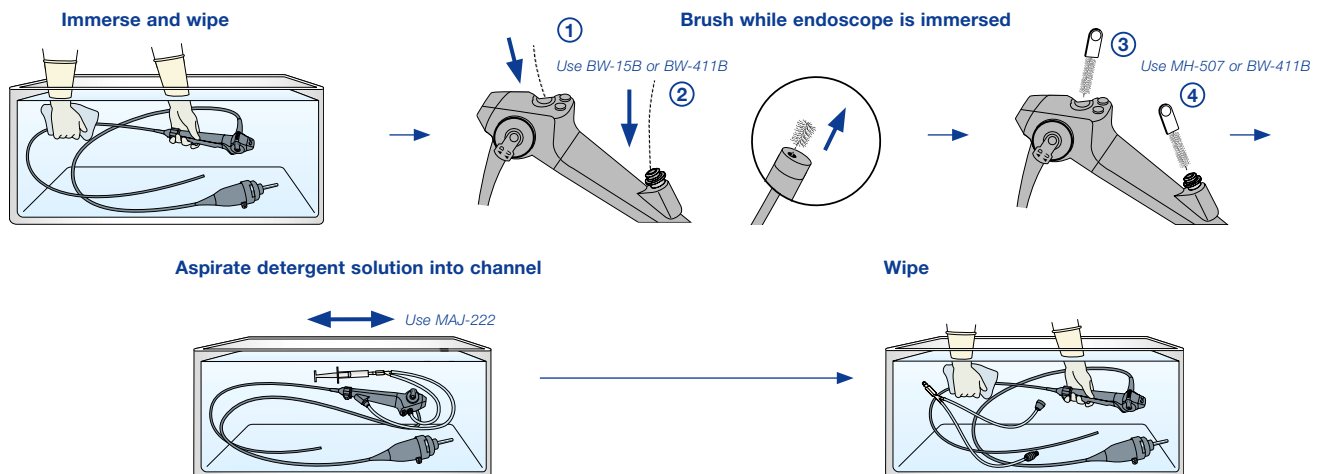

Rinsing

Immerse endoscope and MAJ-222 into water

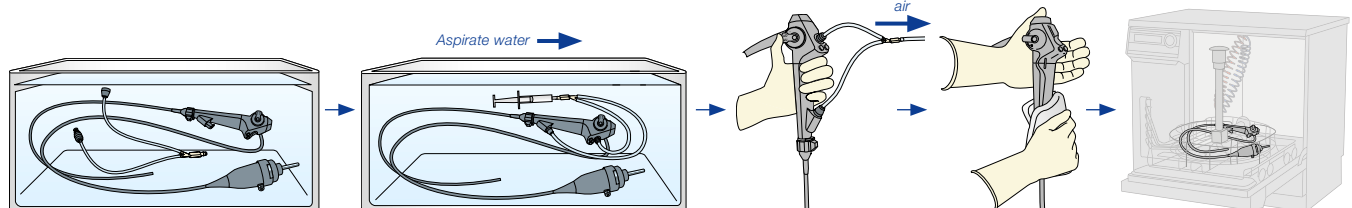

This information does not replace the instructions for use. Please always refer to the instructions given there.

OLYMPUS EUROPA SE & CO. KG

Postbox 10 49 08, 20034 Hamburg, Germany
Wendenstrasse 14-18, 20097 Hamburg, Germany
Phone: +49 40 23773-0, Fax: +49 40 233765
www.olympus-europa.com