

## EZ Clip

**Multiple-Clipping System for Reliable Hemostasis**










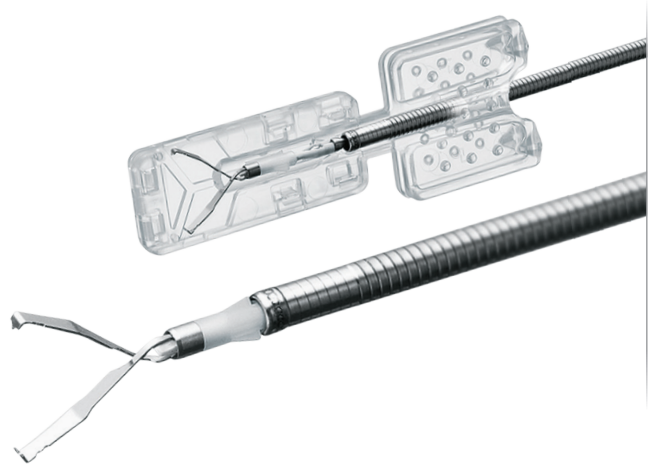

## Clips and Clip-Fixing Devices for Multiple Clipping

**Olympus EZ Clip is a well-known system among healthcare professionals for quite some time now. And there's a good reason for that. The system is highly flexible thanks to functions such as clip rotation and a versatile selection of clips to suit the requirements of any clinical situation. Additionally, it is a very cost-efficient system thanks to the unique reloadable design.**

### Clip Features


- Fast and effective mechanical hemostasis
- Eight types of EZ Clips to suit all applications
- Rotatable
- Sterile cartridge-housed clips
- 90° and 135° jaw angles available
- Short tail
- Colored cartridges to easily distinguish each clip
- For these clips, you have the choice between two different clip applicators



Clip Specifications				
Article Number	Clip Arm Length	Benefits		Jaw Angle
<b>N2303830</b>	Super short, approx. 4 mm	For hard tissue and bleeding ulcers		135°
<b>N1085830</b>	Short, approx. 6 mm	For stronger clipping/smaller area, hemostasis of stiffer lesions		135°
<b>N1085630</b>	Standard, approx. 7.5 mm	For pathological mucosa, deep ulcerous lesions, cancerous tissue		135°
<b>N2303930</b>	Long, approx. 9 mm	Wide clip opening, for post-EMR and post-polypectomy clipping in polyps with big stalk, for tangential approach		135°
<b>N1085730</b>	Short, approx. 6 mm	For marking		90°
<b>N1085530</b>	Standard, approx. 7.5 mm	For healthy, elastic mucosa; Mallory-Weiss syndrome; open vessels; small perforations; marking		90°
<b>N1085930</b>	Long, approx. 9 mm	For healthy, elastic mucosa; large-scale lesions; large adenomatous pedicules before and after polypectomy		90°
<b>N1086030</b>	Short, approx. 6 mm	Special clips for marking; colored distal part; 8 white, red, and yellow clips in each package		90°

### Easy Loading with Simple Push-Pull Action

The clips are housed in cartridges for fast, easy loading. All you have to do is insert the sheath into the cartridge, push the slider forward, and pull back.

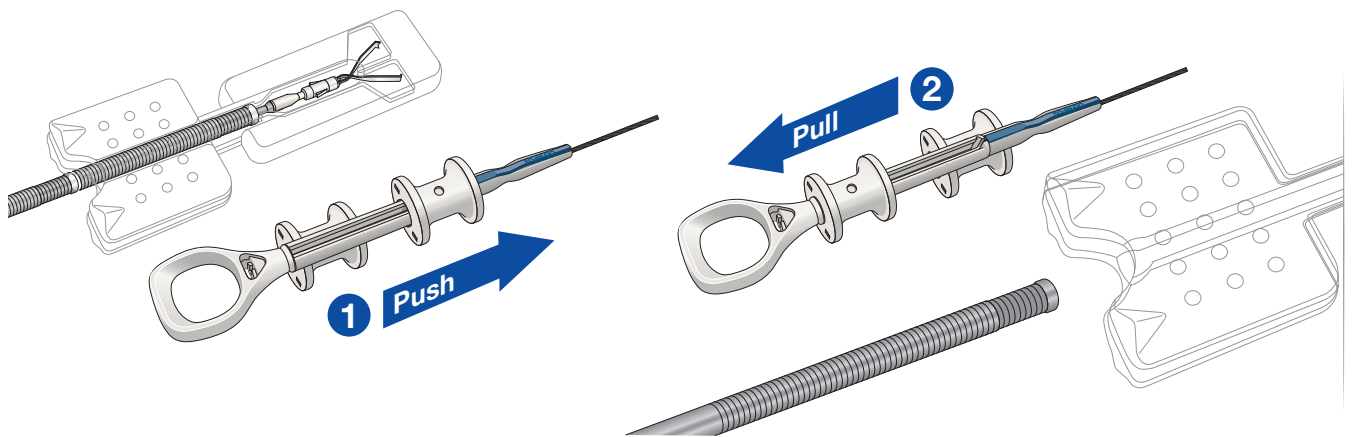


### Applicator Features

Easy clip reloading (push-pull action) with the clip-fixing devices makes it possible to perform multiple clippings smoothly and efficiently during one clinical case.

- Rotation function for easy targeting
- Compatible with all EZ Clip products (HX-610 series)
- Three working lengths available


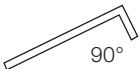
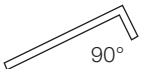
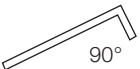
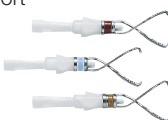




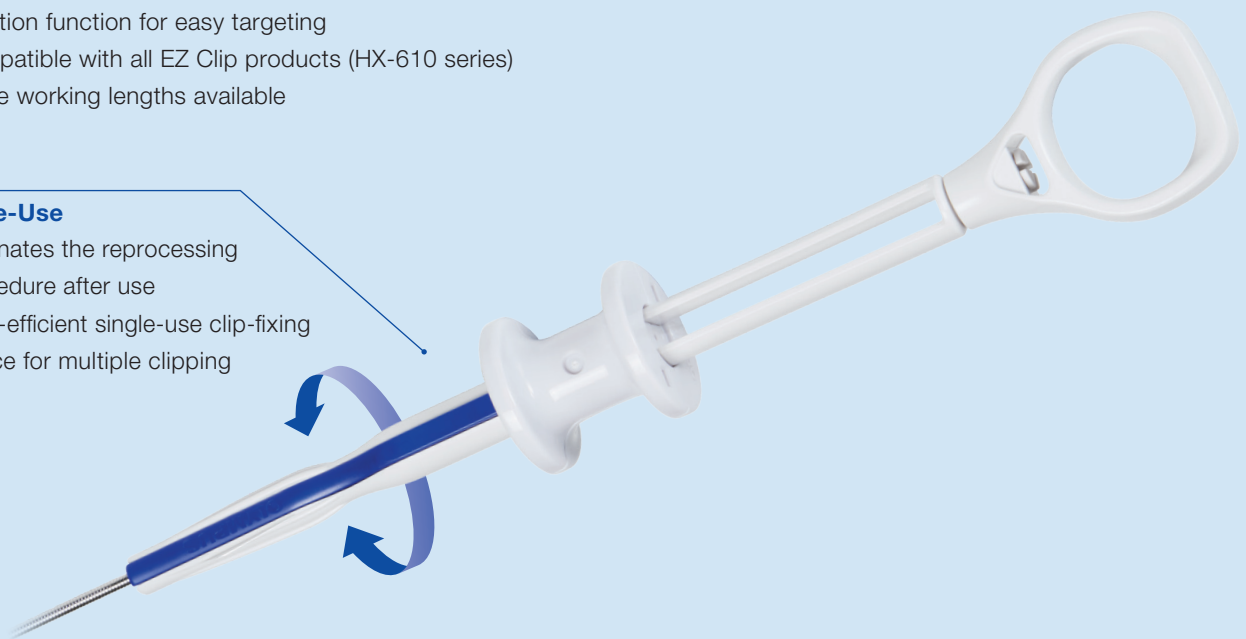


### Single-Use













- Eliminates the reprocessing procedure after use
- Cost-efficient single-use clip-fixing device for multiple clipping

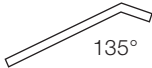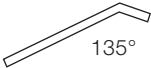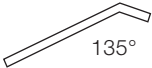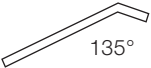

## Clip Specifications

## Super Short (4 mm) / Short (6 mm) / Standard (7.5 mm) / Long (9 mm)

Article Name	HX-610-090SC	HX-610-090S	HX-610-090	HX-610-090L
Article Number	N1086030	N1085730	N1085530	N1085930
Exterior of Jaw Angle				
Clip Arm Length	Short 	Short 	Standard 	Long 
Color Mode	 White	 White	 Yellow	 Blue
Quantity per Box	24	40	40	40

Article Name	HX-610-135XS	HX-610-135S	HX-610-135	HX-610-135L
Article Number	N2303830	N1085830	N1085630	N2303930
Exterior of Jaw Angle				
Clip Arm Length	Super Short 	Short 	Standard 	Long 
Color Mode	 Gray	 Green	 Pink	 Purple
Quantity per Box	24	40	40	40

## Clip-Fixing Device Specifications

## Single-Use Applicator



Article Name	HX-810LR	HX-810QR	HX-810UR
Article Number	N5432730	N5432830	N5432930
Compatible Channel	2.8 mm or more		
Working Length	1650 mm	1950 mm	2300 mm
Maximum Diameter	2.75 mm		
Quantity per Box	10	10	10

 [www.olympus.eu/et-catalog](http://www.olympus.eu/et-catalog)

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.